





WRIST SPLINT AND WRIST AND FOREARM SPLINT

Single Patient Use Only

Rx Only

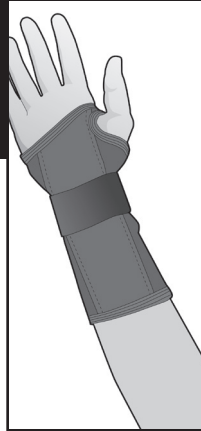

Wrist Splint

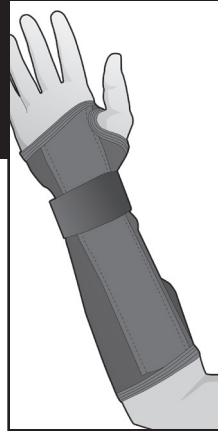

Wrist and Forearm Splint

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of product.

INTENDED USE

The DeRoyal® Wrist Splint is used for carpal tunnel syndrome, tendonitis, sprains and strains of the wrist, and overuse injuries.



The DeRoyal® Wrist and Forearm Splint is used for carpal tunnel syndrome, sprains and strains of the wrist, post-operative immobilization and rehabilitation, and stable fractures of the distal radius and carpal joints.

PRECAUTIONS

This product is to be fitted by a physician (or properly licensed practitioner) who is familiar with the purpose for which they are responsible. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.


INSTRUCTIONS FOR USE

1. Open all hook and loop closures and position splint with palmar side of the splint below the palmar crease. Determine if metal palmar stays offer proper positioning.
2. Remove splint if wrist positioning needs to be adjusted. Adjust metal palmar stay by bending to achieve proper positioning and fit.
3. Reapply splint with palmar side of the splint below the palmar crease.
4. Comfortably secure the hook and loop straps on the dorsal side of the splint.
5. Secure the D-ring strap around the wrist. (If applicable)

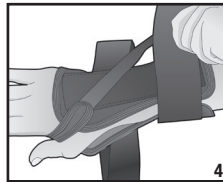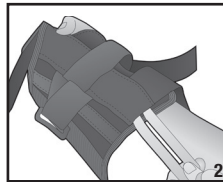

Improving Care. Improving Business.

6. Trim excess strapping where needed.

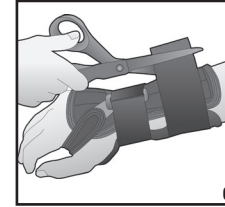

Improving Care. Improving Business.

Manufactured by DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

CARE INSTRUCTIONS

Hand wash lukewarm water, mild detergent, air dry.

WARRANTY

DEROYAL PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

©2011 DeRoyal Industries, Inc. All Rights Reserved.
0-1810 Revised 9/11