



Improving Care. Improving Business.®

WIPEABLE GAIT BELT

	NON-STERILE
	MAGNETIC RESONANCE SAFE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION


Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any product.

INTENDED USE

The DeRoyal® Gait Belts are intended to be prescribed by a properly licensed practitioner to provide caregivers a secure point to hold while assisting patients with walking and/or transfer.





CAUTION

- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this product must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber. Use only as directed.
- **DO NOT** attempt to use this device on an incoherent patient.
- **DO NOT** apply the device where any I.V. or wound site could be compromised.
- Inspect product for damaged or missing components before use. Discontinue use and consult your prescriber if the product or its components break, become damaged or if cleaning is insufficient.
- Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.**
- Consult your prescriber immediately if you experience sensation changes, unusual reactions. Swelling or increased pain while using this product. Discontinue use if pressure injuries develop.

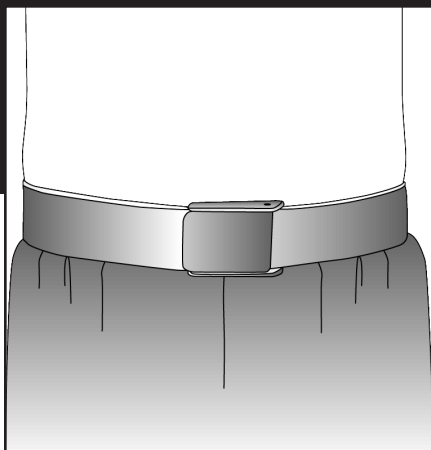

INSTRUCTIONS FOR USE

1. Wrap the belt around the patient’s waist with the buckle closure in front.
2. Feed the end of the belt through the buckle and secure buckle closure. Make sure the buckle lever is pressed down completely.

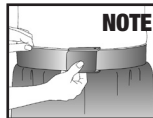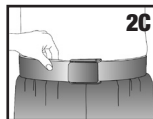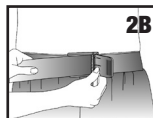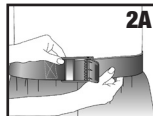

QUICK-RELEASE BUCKLE CLOSURE

Insert the male end into the female end of the buckle. Pull excess strap to adjust waist circumference.

NOTE: The product should fit snug, but remain comfortable.

CLEANING INSTRUCTIONS

Use hospital approved cleaning wipes. Thoroughly wipe all surfaces of device. Allow treated surfaces to remain wet for a full 2 minutes. Air dry completely prior to use. If the inside of the buckle becomes contaminated, discard.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.



Improving Care. Improving Business.®



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#0-1819 Revised 3/2021

©2021 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving
Care. Improving Business. are trademarks or registered
trademarks of DeRoyal Industries, Inc.