**MANUFACTURED BY:**  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
www.deroyal.com

**MedEnvoy Global B.V.**  
Prinses Margrietplantsoen 33 – Suite 123  
2595 AM the Hague  
The Netherlands

**UK Responsible Person**  
MedEnvoy UK Limited  
85, Great Portland Street, First Floor  
London, W1W 7LT  
United Kingdom

**PART#0-1839G | REVISED 10/23**

©2023 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.

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**EN: ENGLISH COVERED BED RAIL PADS**

	<b>SINGLE PATIENT USE</b>
	<b>NON-STERILE</b>
	<b>MEDICAL DEVICE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

**IMPORTANT INFORMATION**

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

**INTENDED USE**

The DeRoyal® Covered Bed Rail Pads are intended to be used to help reduce the risk of bed entrapment, help prevent bed exit and help protect against side rail injuries.

**CONTRAINDICATIONS**

- Do not use on a patient who is or may become highly aggressive or agitated.

**WARNINGS**

- Improper application or use of device may result in serious injury or death.

**CAUTIONS**

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this product must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Inspect product for damaged or missing components before use.
- Discontinue use and consult your prescriber if the product or its components break or become damaged, or cleaning is no longer sufficient to sanitize product.

**INSTRUCTIONS FOR USE**

- Place the bed rail pads on the inside of each side rail or head/foot board.
- Secure the hook and loop closures or elastic straps around the side rails.

<b>SIDE RAIL PAD DIMENSIONS</b> 69"L x 12"H	<b>HEAD/FOOT BOARD PAD DIMENSIONS</b> 32"L x 12"H
--	--

**CLEANING AND/OR MAINTENANCE**

Hand wash with a mild detergent in lukewarm water. Air dry prior to use. Inspect the product after cleaning to ensure it is intact. If the product is damaged or cleaning is insufficient, do not use it and replace it with a new one. Please properly dispose of the product per facility's policies and procedures.

**STORAGE AND TRANSPORT CONDITIONS**

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

**WARRANTY**

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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