DOUBLE-STRAP SECURITY CUFFS

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6. Apply the white strap near the head of the bed and the black strap toward the foot of the bed.

SIZING
Cuff Dimensions
14" L x 3" W (35 x 7.5 cm)
14" L x 3" W (35 x 7.5 cm)

Strap Length
30" (76 cm)
52" (132 cm)

CLEANING AND/OR MAINTENANCE
Prior to washing, remove any detachable rigid stays, fasten any buckles and tie any laces to minimize damage during wash and dry cycles. Close any hook and loop fasteners before washing to help prevent lint build-up and pilling. For maximum product life and to help prevent any straps from tangling, wash in a mesh laundry bag. Hand or machine-wash with a mild fabric detergent in warm water up to 60°C / 140°F. Bleach as directed on container. Do not iron. Lay flat and open and allow to air dry, or tumble dry on low heat for up to 40 minutes. Inspect the product after washing to ensure it is intact and replace any previously removed stays. Hook and loop fasteners may collect lint after repeated use or laundering, reducing grip strength. As needed, use a stiff bristle brush to remove lint from the "hook" side. Test hook and loop adhesion before use and discard product if hook and loop does not hold securely. During the washing, metal buckles could bend and plastic quick-release buckles could crack or break. Always ensure that all buckles are in good working order and are not bent, cracked or broken before each use. If the product is damaged, do not use it and replace it with a new one. Customers could expect up to 5% shrinkage in cotton materials over the life of the product.

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY
DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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TAILLE
Dimensions du brassard
14" Long. x 3" Larg. (35 x 7.5 cm)
14" Long. x 3" Larg. (35 x 7.5 cm)

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DOUBLE-STRAP SECURITY CUFFS

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION
Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE
The DeRoyal® Double-Strap Security Cuffs are intended to be prescribed by a properly licensed practitioner to protect the patient from harming themselves, other patients, family members and/or staff.

CONTRAINDICATIONS
Never use on a patient if an I.V. or wound site could be compromised by the device.

WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Secure straps to a portion of the bed or gurney that moves with the patient. Do not secure straps to bed or gurney side rails.
- Keep straps out of patient's reach. Patient should not be able to remove device.
- Make sure bed or gurney side rails are up while product is in use
- Check the patient regularly to ensure circulation is not compromised.

CAUTIONS
This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use. Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

- Wrap the neoprene cuff around the patient's wrist or ankle making sure the cuff closure is on the outside of the device.
- Secure the cuff with hook and loop closures.
- Secure the straps to the bed using the two-point attachment method with quick release buckles. Release the female end of the buckle and wrap its strap around the anchor.
- Pull the female portion through the loop and connect the male and female ends of the buckle.
- Pull the excess strap to tighten. **NOTE:** Secure excess strap from patient's reach.

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All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.