### STOCKINETTE

### STOCKINETTE

### STOCKINETTE

### STOCKINETTE

### STOCKINETTE

### INVÓLUCROS (ESTOQUINETES)

### STOCKINETTE

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### Explanation of Symbols

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### Explication des symboles

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**Rx Only Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).**

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**Warnings**  
DO NOT reuse. This product is single patient use.

**Intended Use**  
The DeRoyal® stockinette is intended for use to maintain the sterile field on variety of surgical procedures.

**IMPORTANT INFORMATION**  
Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of product.

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**Directions**  
1. Remove sterile stockinette from packaging.  
2. Apply product to arm or leg.  
3. Unroll stockinette past the surgical site or just below it depending on surgeon's preference.

**Storage Conditions**  
Do not store product at extreme temperatures or in a moist/damp environment.

**Sterilization Information**  
Non-sterile product: DeRoyal intends that non-sterile products for use in sterile environments be further processed by our customers, including further packaging or sterilization according to their own validated processes. These products are to be sterilized by ethylene oxide only, as other sterilization methods have not been validated by DeRoyal and may damage the product which could cause a device malfunction or injury to the patient.

**Warnings**  
DeRoyal products are warranted for one hundred and twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship.

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**Precautions**  
This device is for single patient use only. It is not intended for disinfection and subsequent re-use, which may result in device failure or create the risk of contamination.

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

OF MERCHANTS, INCLUDING WARRANTIES PARTICULAR PURPOSE. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

**Warranty**  
DeRoyal products are warranted for one hundred and twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship.

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All Rights Reserved. DeRoyal and the DeRoyal logo are registered trademarks or trademarks of DeRoyal Industries, Inc.