MONITOR SENZA FILI DEROYAL®

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
- To select a different melody alarm tone, open the battery compartment and move the switch to the desired setting. There are three melody options.
- To adjust the volume, turn the volume dial located in the inside of the battery compartment.

OPTIONAL ITEMS

- AC Adaptor can be purchased separately.
- Nurse call port can be purchased separately if the patient's room does not supply one that accommodates this device.

SIZING

Universal

CARE INSTRUCTIONS

Wipe clean with disinfectant wipes. After cleaning, test device before each use.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

- Place cordless bed and/or chair pad(s) within reach. Monitors can accommodate up to two cordless, bed and/ or chair pads (does not accommodate two chair pads).
- Unsnap the flap on the neck of the pad and turn the transmitter switch to the on position. Close the flap and secure the closure.
- Press the program button located on the back of the monitor and hold until the monitor beeps three times.
- Put pressure on the sensor pad to activate the pad and fall monitor. **NOTE:** Use a flat, hard surface to activate the pad and monitor. The fall monitor will beep two times. Press the reset button to activate the system.
- The "Pad 1" light will blink while pressure remains on the pad.
- If the "Pad Signal Lost" light is on and the system does not test properly, reduce the distance between the fall monitor and sensor pad. The ideal distance between the monitor and sensor pad is 15 to 20 feet.
- Test the system before use.

ACTIVATING TWO SENSOR PADS

1. If activating a second pad, activate the first pad as instructed. DO NOT PRESS THE RESET BUTTON. The "Pad 2" light will illuminate. Put pressure on the second pad and release. The fall monitor will beep two times again and the light will turn off. At this time, press the reset button and both pads will be activated.

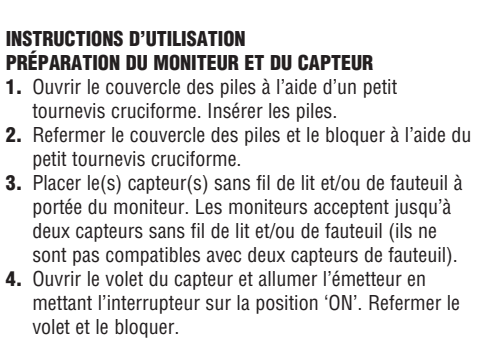
MOUNTING THE MONITOR

- The strap with buckle can be used to attach the monitor to a desired location.
- To mount the monitor, mount the screw onto the wall or headboard. Hang the monitor on the screw using the mounting hole on back of the monitor.

TURNING OFF OR RESETTING THE ALARM

- When the alarm is triggered, the alarm can be turned off and reset with either the reset button or Cargiver Key™ function.
- Select either "Reset Button ON" or "Caregiver Key™ ON" with the switch on the side of the monitor. Use the end of a pen to move the switch.
- If set on "Reset Button ON" position, the caregiver can reset the monitor by pressing the reset button located on the front of the monitor.
- If set on the "Caregiver Key ON" position, the monitor can only be reset by using the Caregiver Key provided with the monitor. The Caregiver Key must be placed over the red dot to reset the monitor.

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DEROYAL® CORDLESS MONITOR

RX ONLY
REUSABLE
NON-STERILE
NOT MADE WITH NATURAL RUBBER LATEX

IMPORTANT INFORMATION

Please read all instructions, warnings and precautions before use. Correct set up is essential for proper functioning of product.

This product is to be used by a licensed caregiver only for the use it was intended.

INTENDED USE

The DeRoyal Cordless Monitor is intended to alert staff and patients in the event the patient attempts to exit the bed, wheelchair and/or recliner.

WARNINGS

Failure to comply with these warnings may result in injury or death. This device is not suitable for all individuals. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive patient mobility management program. Test this device before each use. Do not use without reading the instructions.

This monitor is to be installed by a licensed caregiver. As such, it is the entire responsibility of the caregiver to ensure that the system is not designed to replace good caregiving practices including but not limited to:

- Direct patient supervision
- Adequate training for staff personnel for fall prevention and/or elopement
- Testing of the system before each use

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

SETTING UP THE MONITOR AND SENSOR PAD

- Open battery compartment door with a small Phillips-head screwdriver. Insert batteries.
- Close the battery door and secure it with the small Phillips-head screwdriver.

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