**Explanation of Symbols****Explicación de los símbolos****Explication des symboles****Erläuterung der Symbole****Spiegazione dei simboli****Explicação dos Símbolos****Verklaring van de symbolen****Symbolförklaringar****Условные обозначения****Επεξήγηση των συμβόλων****Rx Only**

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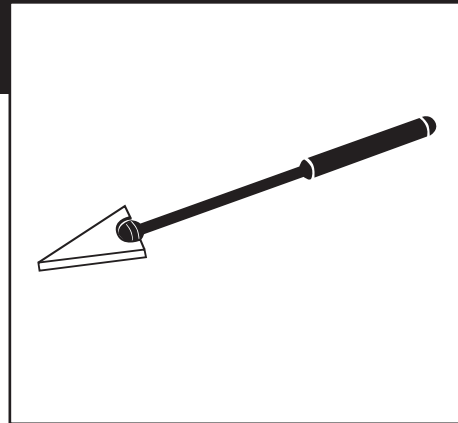
Improving Care. Improving Business.

Manufacturer:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
988.938.7828 or 865.938.7828  
www.deroyal.com  
Distributed in the US & Worldwide  
by DeRoyal Industries, Inc.

QNET BV  
Hommertweg 286  
6436 AM Amstenrade  
The Netherlands

**Part #0-1896G**  
Revised 11-2014  
©2014 DeRoyal Industries, Inc.  
All Rights Reserved.

**CE**  
**0543**  
10/2014

**PVA EYE SPEARS****ESPONJAS OFTALMOLÓGICAS DE PVA****LANCETTES OCULAIRES PVA****PVA-AUGENTUPFER, SPEERFORM****LANCETTE ASSORBENTI PVA****LÂMINAS OFTÁLMICAS DE PVA****PVA-PILTORKAR FÖR ÖGON****PVA LANCETDEPPERS****ТУПФЕРЫ ОФТАЛЬМОЛОГИЧЕСКИЕ ИЗ ПОЛИВИНИЛАЦЕТАТА (ПВА)****OI PVA EYE SPEARS****PVA EYE SPEARS****DO NOT REUSE  
DOES NOT CONTAIN NATURAL RUBBER LATEX****RX ONLY****IMPORTANT INFORMATION**

Please read carefully all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

**DESCRIPTION**

The PVA Eye Spears consist of an absorbent polyvinyl acetal (PVA) spear attached to a high density plastic handle.

**INDICATIONS**

The PVA Eye Spears are designed for use by properly licensed medical professionals to absorb fluids and for removal of debris from the operative field or instruments during ophthalmic surgery.

**CONTRAINDICATIONS**

Patients with allergies to any of the components should refrain from using this product.

**INSTRUCTIONS FOR USE**

1. Check that the individual sterile protective package is unopened and verify the expiration date.
2. Open the individual sterile protective pack aseptically.
3. Remove the eye spears aseptically and place on the operating table.
4. Use the eye spears to absorb fluids and to remove debris from the operative field or instruments during ophthalmic surgery.
5. Do not use in the dry state for refractive surgery (Lasik, etc.)
6. Do not resterilize. At the end of surgery, place the eye spear in disposable waste bags.

**DISPOSAL CONDITIONS**

Waste generated: The waste generated is considered as waste involving infectious or related hazards. There are two types of waste material: polyvinyl acetal and polypropylene.

**STORAGE**

Store at ambient temperature and protect from moisture.

**WARRANTY**

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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