## SutureClose™ LAPAROSCOPIC PORT CLOSURE SYSTEM

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## SutureClose™ LAPAROSCOPIC PORT CLOSURE SYSTEM

⊗	<b>DO NOT REUSE</b>
STERILE/EO	<b>STERILIZED WITH ETHYLENE OXIDE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
RX ONLY	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

#### IMPORTANT INFORMATION

Please read all warnings and instructions before use.

#### DESCRIPTION:

The DeRoyal® SutureClose™ Laparoscopic Port Closure System, is a sterile and single-use product. It incorporates a 5mm cone-shaped needle introducer and a 15 cm needle with retractable jaws. The device has application in laparoscopic procedures for approximation of tissues and percutaneous suturing for closing incision sites. The device must be used with a suture of appropriate size to close the incision site.

#### CONTRAINDICATIONS:

- Do not use where laparoscopic techniques are generally contraindicated.
- Do not use SutureClose™ where it is not possible to visualize the tip of the device during insertion or removal.

#### ⚠️ WARNINGS AND PRECAUTIONS:

- Federal (U.S.A) law restricts this device to sale, distribution, and use by or on the order of a physician.
- This product is EO sterilized and for single use only. Please do not re-use or re-process to avoid irreversible injury and device malfunction.
- Do not use the product if the package is opened or damaged.
- Store this device in a dry, clean and safe place. Keep the device away from flammable and combustible chemicals and materials.
- Please follow universal guidelines upon medical waste disposal to avoid any potential cross-infection, environmental contamination or injury caused by sharps waste.
- Do not depress the button during insertion of SutureClose™ to do so may prevent the needle tip from penetrating tissue.
- Do not depress the button during removal of SutureClose™ to ensure that the hooked end does not inadvertently catch tissue.
- Always visualize the tip either directly or through a laparoscope during insertion and removal of SutureClose™ to prevent an unintentional puncture of underlying organs.
- After removing SutureClose™ from the abdominal cavity, always inspect the site for hemostasis.

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#### INSTRUCTIONS FOR USE:

- Insert the needle introducer with the holes aligned vertical to the trocar wound. Depress the top button to expose the jaws inside the stylet and load the suture. Puncture the needle through fascia, muscle and peritoneum into the abdomen. Drop the suture and remove the SutureClose™ needle.
- Insert the SutureClose™ needle through the other hole of the needle introducer. Puncture the SutureClose™ needle through all the layers and pick up the suture with the jaws of SutureClose™ needle.
- Pull the suture up through the peritoneum, muscle, fascia and needle introducer.
- Remove the needle introducer and tie off the suture.

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

#### WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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©2021 All Rights Reserved.
trademarks or registered trademarks of