



Improving Care. Improving Business.®

## DEROYAL® LATCHING SAFETY AUTO RESET MONITOR

	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

### IMPORTANT INFORMATION

Please read all instructions, warnings and precautions before use. Correct set up is essential for proper functioning of product.

This product is to be used by a licensed caregiver only for the use it was intended.

### INTENDED USE

The DeRoyal® Latching Safety Auto-Reset Monitor is intended to alert staff and patients in the event the patient attempts to exit the bed, wheelchair and/or recliner. It is also intended to be used with latching nurse call systems.

### WARNINGS

Failure to comply with these warnings may result in injury or death. This device is not suitable for all individuals. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive patient mobility management program. Test this device before each use. Do not use without reading the instructions.

This monitor is to be installed by a licensed caregiver. As such, it is the entire responsibility of the caregiver to ensure that the system is not designed to replace good caregiving practices including but not limited to:

- Direct patient supervision
- Adequate training for staff personnel for fall prevention and/or elopement
- Testing of the system before each use

**DO NOT** connect a DeRoyal® alarm to other manufacturers' sensors. Use of another manufacturers' sensor may damage the alarm, may cause the fall monitoring system not to function as intended, and will void the DeRoyal warranty.

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.



### INSTRUCTIONS FOR USE

#### SETTING UP THE MONITOR AND SENSOR PAD

1. Open battery compartment door with a small Phillips-head screwdriver. Insert batteries.
2. Close the battery door and secure it with the small Phillips-head screwdriver.
3. Place one bed sensor pad or chair sensor pad within reach.
4. Connect the monitor to the bed sensor pad or the chair sensor pad. Plug the cord end of the sensor pad into the bottom of the monitor. Pinch the cord together and route it through the cord protection recess on the monitor (near the plug input).
5. Put pressure on the sensor pad to activate the pad and monitor. The monitor will beep two times to indicate that the sensor pad and fall monitor are in use. The "Monitor Status" light will blink while pressure remains on the pad. When pressure is removed, the fall monitor will sound the alarm. Press the "Reset" button to silence the alarm.
6. If the "Pad Lost Light" is on, check to make sure the cord is plugged in and reset the system.
7. Test the system before use.

#### MOUNTING THE MONITOR

1. The strap with buckle can be used to attach the monitor to a desired location.
2. To mount the monitor, mount the screws onto the wall or headboard. Hang the monitor on the screws using the mounting holes on back of the monitor.

#### TURNING OFF OR RESETTING THE ALARM

1. When the alarm is triggered, the alarm can be turned off and reset with either the reset button or Caregiver Key™ function.
2. Select either "Reset Button ON" or "Caregiver Key™ ON" with the switch on the side of the monitor. Use the end of a pen to move the switch.
3. If set on "Reset Button ON" position, the caregiver can reset the monitor by pressing the reset button located on the front of the monitor.
4. If set on the "Caregiver Key™ ON" position, the monitor can only be reset by using the Caregiver Key™ provided with the monitor. The Caregiver Key™ must be placed over the red dot to reset the monitor.



Improving Care. Improving Business.®

### VOLUME, TONE AND DELAY OPTIONS

1. Adjust the volume by turning the dial located inside the battery compartment.
2. Select an alarm tone by adjusting the switch inside the battery compartment to the desired setting.
3. Select alarm tone delay time by adjusting the switch inside the battery compartment.

### OPTIONAL ITEMS

1. AC Adaptor can be purchased separately.
2. Nurse call cord can be purchased separately if the patient's room does not supply one that accommodates this device.

### CARE INSTRUCTIONS

Wipe clean with disinfectant wipes. After cleaning, test device before each use.

### STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

### WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufacturered for:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
[www.deroyal.com](http://www.deroyal.com)

Part#0-1950 Revised 4/19

©2019 DeRoyal Industries, Inc.  
All Rights Reserved.

DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are registered trademarks of DeRoyal Industries, Inc.

Safety Auto-Reset and Caregiver Key are trademarks of Smart Caregiver Corporation.