FOAM ANKLE STIRRUP

ESTRIBO DE ESPUMA PARA TOBILLO

SUPPORT DE CHEVILLE EN MOUSSE

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FOAM ENKELBEUGEL

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KÖPÜK PEDLİ AYAK BİLEKLİĞİ

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Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.derooyal.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Part#0-1960G
Revised 2/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.

FOAM ANKLE STIRRUP

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

INTENDED USE

The DeRoyal® Foam Ankle Brace is used for acute and chronic ankle sprains, osteoarthritis and syndesmosis sprain.





CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber. Use only as directed.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the ankle (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Wear sock underneath this brace.
- Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

INSTRUCTIONS FOR USE

NOTE: A sock should be worn under this brace.

- Spread the shells apart and place the patient's heel on the heel strap (located between the shells).
- Place the shells against the sides of the ankle.
- a.) Compress the shells around the patient's ankle by tightening straps in opposing directions, while securing the straps from bottom to top. b.) **NOTE:** To adjust heel strap for a snug fit, raise or lower the heel straps located on the outer side of the shells. Trim heel straps as needed.
- Check for adequate circulation. Readjust straps as needed for a secure and comfortable fit.

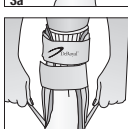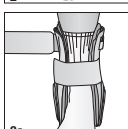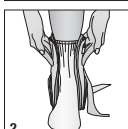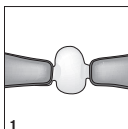CARE INSTRUCTIONS

Shell: Wipe down with damp cloth. Air dry. Sleeve: Detach foam cushion from brace. Remove foam from sleeve. Machine wash sleeve with mild detergent. Tumble dry low.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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