



Improving Care. Improving Business.®

## FACE SHIELD W/ ELASTIC STRAP

	<b>DO NOT REUSE</b>
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

### IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product. Read carefully and completely before using or applying.

### DESCRIPTION

The face shield is a plastic, optically clear, anti-fog coated shield with a solid foam strip held in place with an adjustable soft elastic strap.

### INTENDED USE & INDICATIONS

The DeRoyal® Face Shield is intended for use whenever splashes, spray, splatter or droplets of blood or other bodily fluids may be generated and contamination may be reasonably anticipated.

### WARNINGS

- Wearing this product will **NOT** eliminate exposure or risk of contracting any disease or infection.
- Change this product immediately and dispose of properly if damaged or contaminated with blood or bodily fluids.
- Device must be used in conjunction with appropriate nose and mouth mask.
- This product **DOES NOT** meet ANSI Standard Z87.1.
- **DO NOT** clean or re-sterilize.
- This device is **NOT** designed, sold or intended for use except as indicated.
- This product is **NOT** impact resistant and should not be worn as protection from solid objects.
- **DO NOT** use as a shield against acids, alkaline and other hazardous chemicals known to attack polyester and other films.
- **DO NOT** use as protection against a strong light source, gases or vapors of any kind, sparks, radiation, ultraviolet rays or lasers.
- **DO NOT** use while smoking or performing any activities involving the use of or proximity to flame or heat-producing equipment.

### DIRECTIONS FOR USE

1. Place face shield over head with plastic film in front of face and strap around head.
2. Foam spacer should be positioned on forehead.
3. When in use, **DO NOT** look around or under shield. Ensure shield is always in front of face.
4. Dispose of used shield according to facility protocol.

### STORAGE AND TRANSPORT CONDITIONS

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

### WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufacturerd by:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
www.deroyal.com

### PART#0-1985 REVISED 3/2020

©2020 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.