






PEACH CLIP

	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

 Please read all warnings and instructions before use.

INTENDED USE

The Peach Clip is used to attach surgical drapes to IV poles to create a barrier at the patient's head and can be used for a variety of other clamping purposes.

WARNINGS

- For use on equipment only. **DO NOT** use on patients.
- Clean after use.

CLEANING INSTRUCTIONS

1. Open clip to inspect the spring, hinge points, teeth, and all surfaces of the clip to ensure clip functions properly and is free of damage and contamination.
2. If clip does not function properly, is damaged, visibly soiled, or is known to have come into contact with bodily fluids or other tissues, dispose of per facility protocol.
3. If the clip is suitable for reprocessing, use one of the following methods:
 - Preferred method (Sterile): Steam autoclave, maximum of 5 reuses.
 - Preconditioning pulses: 3
 - Temperature: 132 °C
 - Full Cycle Time: 12 minutes
 - Minimum Dry Time: 40 Minutes
 - Alternate method (Non-Sterile): Clean per facility guidelines.
4. Avoid harsh chemicals such as bleach, strong acids, alkalis, or other substances that may damage nylon or stainless steel.



DISPOSAL

Dispose of any used device according to local, state, and federal laws and regulations. For safe disposal of devices, follow your facility's protocol.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for ninety (90) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#0-1999 | REVISED 8/2025

©2025 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal and the DeRoyal logo are registered trademarks of DeRoyal Industries, Inc.