

Step 1: Assemble Liner



- » Attach self-aligning lid to liner, using moderate pressure
- » Adhesive on lid should adhere to entire circumference of the liner

Step 2: Place Liner in Outer Shell



- » Place liner assembly in outer shell
- » Connect blue **VACUUM** tubing to **VACUUM** port
- » Turn valve to **ON** position

Step 3: Lid Set-Up



- » Secure cap on **TANDEM** and **POUR** Ports
- » Place suction tubing on **PATIENT** Port (*elbow optional*)

Step 4: Liner Disposal



Do not dispose of outer shell when replacing the inner liner

- » Turn vacuum source **OFF** using the **ON/OFF** valve on the outer shell
- » Disconnect tubing from **PATIENT** port
- » Remove blue hose from **VACUUM** port
- » Securely cap all ports
- » Remove liner leaving outer shell in place*
- » Dispose of drainage and disposable liner in compliance with facility's guidelines*

This Assembly Guide does not supersede the instructions given by a prescriber or the information contained in the Instructions for Use ("IFU") which accompany this device. Please refer to the IFU for all warnings, cautions and complete instructions.

The DeRoyal logo, Improving Care, Improving Business, and SafeLiner are registered trademarks of DeRoyal Industries, Inc.