**DeRoyal**<sup>®</sup>

## EN: STEEP-T<sup>®</sup> MULTI-ANGLE PATIENT POSITIONING SYSTEM

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**MANUFACTURED BY:**  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
www.deroyal.com

**MedEnvoy Global B.V.**  
Prinses Margrietplantsoen 33 – Suite 123  
2595 AM the Hague  
The Netherlands

**UK Responsible Person**  
MedEnvoy UK Limited  
85, Great Portland Street, First Floor  
London, W1W 7LT  
United Kingdom

### PART#0-2340G | REVISED 4/26

©2026 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Steep-T are registered trademarks of DeRoyal Industries, Inc.

Dispatch is a registered trademark of The Clorox Company

## STEEP-T<sup>®</sup> MULTI-ANGLE PATIENT POSITIONING SYSTEM

	<b>SINGLE PATIENT USE</b>
	<b>NON-STERILE</b>
	<b>MEDICAL DEVICE</b>
	<b>DO NOT USE IF PACKAGE IS DAMAGED</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

### IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of product.

### INTENDED USE

The STEEP-T<sup>®</sup> Multi-Angle Patient Positioning System is intended to help prevent unwanted movement when patients are in steep-angled positions while protecting patients from nerve injuries, potential pressure injuries, and/or tissue breakdown.

### CONTRAINDICATIONS

- **DO NOT USE** on patients over 500lbs.

### WARNINGS

- Prior to use, inspect the kit components for damaged, missing, or contaminated components.
- **DO NOT** attempt to carry patients while using this device.
- Use only as directed.
- Ensure kit components are installed correctly and the patient is properly positioned on the device to prevent movement.
- **DO NOT USE** the kit without all components included in the kit.
- Discontinue use if skin reaction occurs.

### CAUTIONS

- The components within this system have only been tested in conjunction with each other. Use of outside components could alter proper performance.
- Ensure the OR table pad is attached securely to the OR table and is clean and free of residue.
- Be sure to follow your hospital's policies and procedures for patient monitoring.
- Discontinue use if device does not properly support patient.

### DIRECTIONS FOR USE

#### PREPARE THE OR TABLE

1. Remove the current back/seat section w/perineal cutout pad from the OR table.
2. Replace with the provided back/seat section w/perineal cutout pad designed for our system, ensuring it is attached securely to the OR table.

**APPLYING PATIENT POSITIONING SYSTEM WITH ARM PROTECTION WEDGE**

1. Place the patient pad with handles and head support pillow into correct position to receive a patient.

2. Utilizing the shorter strap and both sides of the bed rail, secure the patient pad to the OR table pad. Strap location should correspond to the lower lumbar region of the patient. **IMPORTANT: DO NOT** secure strap to bed rails at this time.
3. Transfer patient onto the OR table. If patient needs to be repositioned, use the attached handles to reposition the patient as needed and resecure the table strap. **NOTE:** When repositioning the patient pad ensure the pad is completely lifted off the interlocking OR table pad. The non-skid materials are designed to not slide on one another.
4. Once patient is properly positioned, secure the foam pad to the OR table by tightening straps to compress the patient pad and OR table pad together.
5. Place the scapular wedges under each shoulder for additional points of fixation. Using one hand to raise the patient's shoulder, place a scapular wedge slightly angled towards the sternum or midline of the body. Repeat with other shoulder.
6. To place the arm wedge, gently roll the patient toward the clinician and place the wedge under the patient with the peak of the wedge aligned with the lateral edge of the torso. Align the top of the device with the middle of the humerus or upper arm. Secure the arm tightly in position with the hook and loop closure system. **NOTE:** The arm securement wedges should cover the table securement strap so it does not contact the patient's skin.
7. Repeat with the other arm.
8. Place the longer chest strap across the patient and secure to the bed railing tautly.
9. Before procedure begins, make a visual assessment to ensure the surgical team is in agreement with final patient positioning.

#### APPLYING PATIENT POSITIONING SYSTEM WITH ARM PROTECTION PADS

1. Place the patient pad with handles and head support pillow into the correct position to receive a patient.
2. Utilizing the shorter strap and both sides of the bed rail, place the strap across the patient pad. **NOTE:** The location of the strap should correspond to the lower lumbar region of the patient. **IMPORTANT: DO NOT** secure strap to the bed rails at this time.
3. Lay the lift sheet over the patient pad and the table strap.
4. Transfer patient onto OR table and properly position patient on pad. If the patient needs to be repositioned, utilize the lift sheet to carefully lift the patient up and off the pad to reposition as needed.
5. Once the patient is properly positioned, secure the foam pad to the OR table by tightening the straps to compress the patient pad and OR table pad together.
6. Wrap the foam arm protectors around the patient's arms and secure tightly using the hook and loop straps. Ensure the patient's fingertips are covered by foam.
7. To secure the patient's arms, pull the lift sheet up between the patient's arms and torso, wrap around the arm protectors, and then tuck underneath the patient's torso.
8. Place the longer strap across the patient's chest and secure to the bed railing tautly.
9. Before the procedure begins, make a visual assessment to ensure the surgical team is in agreement with final patient positioning.

#### CLEANING INSTRUCTIONS

- Follow hospital protocol.
- DeRoyal has validated that the following cleaning agents or

- their equivalents are acceptable for cleaning:
- Hydrogen Peroxide
  - Dispatch<sup>®</sup> Hospital Cleaner Disinfectant Towels with Bleach

#### STORAGE AND TRANSPORT CONDITIONS

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

#### WARRANTY

DeRoyal<sup>®</sup> products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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