


SutureClose™

Laparoscopic Port Closure System

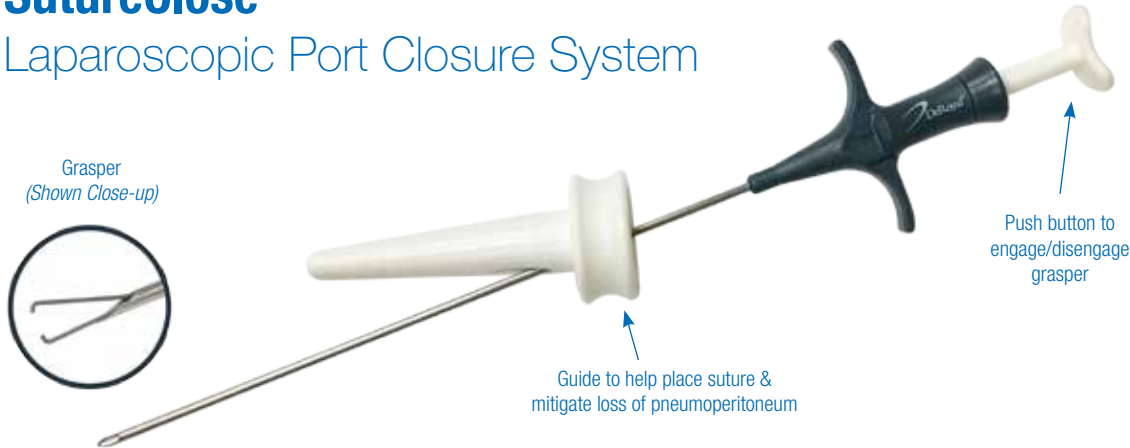

Step1



Step2



Step3



Step4

- Technology for fascia/peritoneum closure
- Ability to close multiple sites during same procedure
- Closure under visualization
- Mitigates the possibility of port site hernias
- Closure without having to extend the incision
- One handed operation
- Sterile and single use device

The DeRoyal® SutureClose™ aids in closing incisions and offers a spring-loaded grasper to help guide the suture when closing the site. With a push of a button, the grasper is engaged and is ready to accept the suture. Once the suture is firmly grasped, and the button is disengaged, the suture retracts into the needle shaft. Simply insert the needle into the peritoneum and release the suture using the needle guide. Reverse the angle of the grasper and retrieve the suture for closure of the incision site.

SutureClose™ Laparoscopic Port Closure System

28-1001 | Laparoscopic Closure Needle and Guide | 10 / Case

DeRoyal, the DeRoyal logo, Improving Care, Improving Business, are registered trademarks of DeRoyal Industries, Inc.

SutureClose is a trademark of DeRoyal Industries, Inc.

Copyright© 2016, DeRoyal. All rights reserved.

DeRoyal Industries, Inc.
200 DeBusk Lane | Powell, TN 37849
1.800.DEROYAL or 865.938.7828
www.deroyal.com | Rev. 08/16 Reprint #0-2399



Improving Care. Improving Business.®