


Comparative Testing Between DeRoyal and Market Leading Positioning Sheets


The PRUventor TurnPRO® positioning sheet is a two-layer laminated fabric. The gray side is a durable, breathable nylon rip-stop. The blue side is a breathable polyester mesh treated with a silver ion based antimicrobial agent.

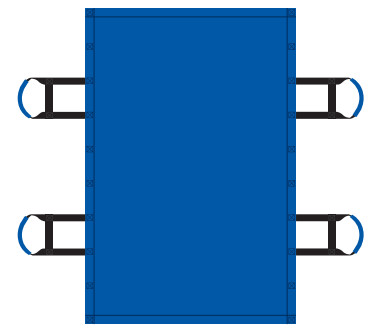

Sheet Features & Benefits

- Antimicrobial treatment minimizes odors
- Breathable material allows moisture vapor and airflow through sheet
- Enhanced moisture vapor transmission rate improves airflow to the patient's skin surface


Moisture Vapor Transmission Rate

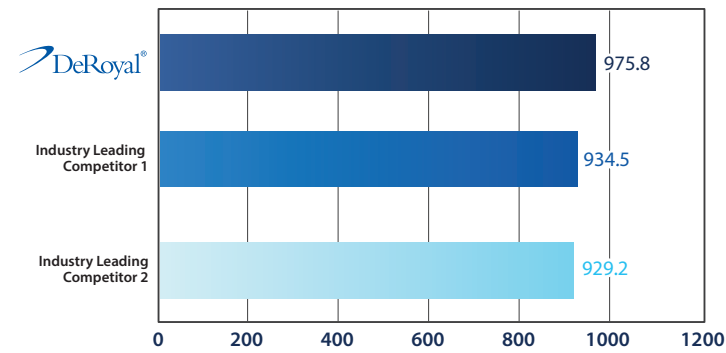

MVTR Average Result (g/m2/24hr)

MVTR is the measure of the passage of water vapor through a material
*data on file

Air Permeability

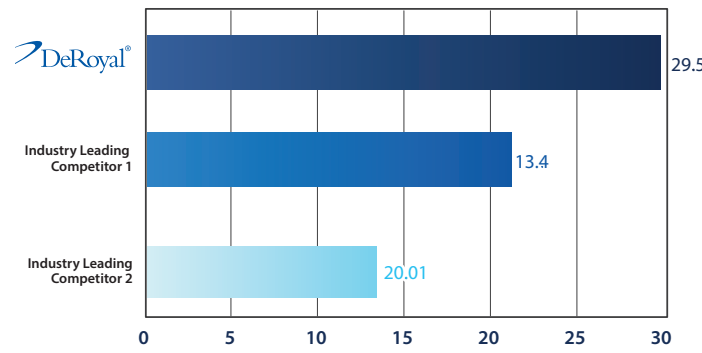

Air Permeability Average Result (cfm)

*data on file

Antimicrobial Effectiveness Testing

Microbial	Test Method	Percent Reduction
Escherichia Coli	AST E2149-20	≥ 99.95%
Staphylococcus Aureus	AATCC TM100	≥ 99.92%
Klebsiella Pneumoniae	AATCC TM100	≥ 99.62%
Candida Albicans	AATCC TM100	≥ 88.20%

*The test was performed for the antimicrobial effectiveness of the blue polyester fabric.
*data on file

PRUventor
TURNPRO

Turning & Positioning System

The PRUventor TurnPRO® System is a **Safe and Easy-To-Use** Patient Handling Solution for Clinicians



PRUventor TurnPRO® System


SHEET

- Soft, strong, ergonomic handles for a comfortable and secure place to hold while turning and positioning the patient
- Breathable material allows moisture vapor and airflow through sheet
- Top handles on sheet for adjusting patient at head of bed
- Labeling to assist in the proper use of the product
- Reinforced border handles offer strap stability and durability
- Less friction for easy repositioning



ABSORBENT PAD

- Soft, breathable, moisture-managing top layer
- Quiet, non-slip back to keep pad in place
- Leakproof bottom layer to help prevent contamination of sheets
- Patented SuperCore® technology provides premium absorbency and moisture wicking capabilities¹

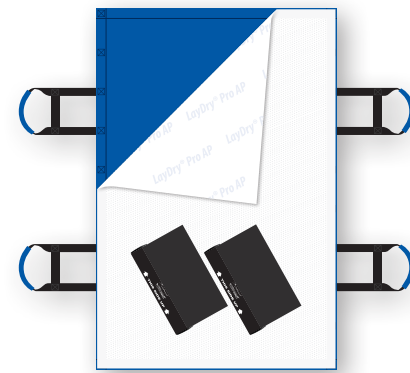

COVERED WEDGES

- High-density foam accommodates varying patient weights
- Standard 30° angle supports off-loading the sacrum
- Fully covered wedges with high grip fabric to prevent migration
- Integrated positioning handles allow easy adjustment of the wedges
- Anti-slip top and bottom surface ensures wedge securement
- Easy-to-clean, wipeable surface on both top and bottom of wedges

Indications: To assist and maintain proper positioning of the patient while helping prevent sacral pressure injuries

Size	Weight Limit	Dimensions	Qty	Product No.
Universal	≤500 lbs.	35" x 55"	5 Systems	M70-500*
Universal	≤500 lbs.	35" x 55"	5 Systems w/ 5 additional pads per system	M70-500A*
X-Large	≤750 lbs.	45" x 55"	5 Systems	M70-600*
X-Large	≤750 lbs.	45" x 55"	5 Systems w/ 5 additional pads per system	M70-600A*



Scan Here To View
TurnPRO System

¹<https://mccairlaidusa.com/products/laydry-adult-bed-pads>


⚠️ WARNING - This product can expose you to chemicals including N,N-Dimethylformamide, which is known to the State of California to cause cancer. For more information, go to www.P65Warnings.ca.gov.



Absorbent Pad

Indications: To help control body heat and manage moisture


Size	Dimensions	Qty	Product No.
Universal	35" x 55"	20 Ea	M70-500IP
X-Large	40" x 55"	20 Ea	M70-600IP

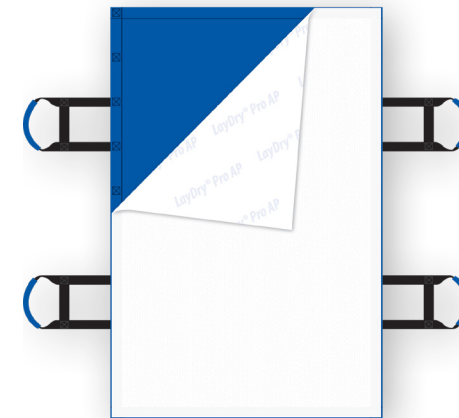

Sheet w/ Absorbent Pad

Indications: To provide assistance to the caregiver when repositioning the patient; To help control body heat and manage moisture

Size	Dimensions	Qty	Product No.
Universal	35" x 55"	10 Sets	M70-500TS
X-Large	45" x 55"	10 Sets	M70-600TS

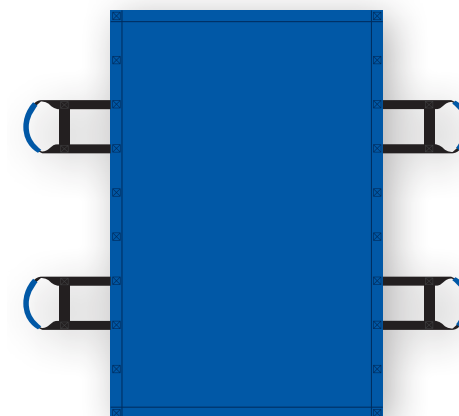

Sheet w/o Absorbent Pad

Indications: To provide assistance to the caregiver when repositioning the patient.

Size	Dimensions	Qty	Product No.
Universal	35" x 55"	10 Ea	M70-500S
X-Large	45" x 55"	10 Ea	M70-600S



Covered Wedges

Indications: To off-load the sacrum; To help maintain proper positioning

Size	Dimensions	Qty	Product No.
Universal	16" x 9" x 5"	5 Pr	M70-500CPW*
X-Large	16" x 14" x 8"	5 Pr	M70-600CPW*



Wedges

Indications: To off-load the sacrum; To help maintain proper positioning

Size	Dimensions	Qty	Product No.
Universal	16" x 9" x 5"	5 Pr	M70-500PW*
X-Large	16" x 14" x 8"	5 Pr	M70-600PW*

⚠️ WARNING - This product can expose you to chemicals including N,N-Dimethylformamide, which is known to the State of California to cause cancer. For more information, go to www.P65Warnings.ca.gov.

