EN: I.V. ARM BOARD

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FR: ACCOTOIR I.V.

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IT: REGGI-BRACCIO PER TERAPIA E.V.

PT: PLACA DE BRAÇO I.V.

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MANUFACTURED BY:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33 – Suite 123
2595 AM the Hague
The Netherlands

UK Responsible Person
MedEnvoy UK Limited
85, Great Portland Street, First Floor
London, W1W 7LT
United Kingdom

PART#0-601G I REVISED 10/23

©2023 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.

EN: ENGLISH I.V. ARM BOARD

	SINGLE PATIENT USE
	NON-STERILE
	MEDICAL DEVICE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product.

INTENDED USE

Positioning aid for I.V. therapy; Arterial support

CAUTIONS

- This product is to be fitted by a physician (or properly licensed practitioner) that is familiar with the purpose for which they are responsible. The physician or practitioner is responsible for providing use instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- Follow good nursing and basic patient care practices and other recommended practices of your health care facility.
- Follow the instructions of physician (or properly licensed practitioner) for length and duration of use.

SIZING

Armboards are available in infant, child, and adult sizes.

CAUTIONS

- DO NOT** over-tighten so that the straps restrict the patient's circulation.

DIRECTIONS FOR USE

- Open straps on I.V. Board.
- Conform board by bending into desired position.
- Place patient's wrist on I.V. Board (palm up for arterial cannulation).
- Bring straps over patient's wrist/hand in desired locations. Straps are secured by firmly attaching to hook on the back of the board.
- Trim excess strapping as needed. **DO NOT** over-tighten so that the straps restrict the patient's circulation.

CARE INSTRUCTIONS

This product is disposable and should not be laundered or reused. Discard after use.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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