




## Telemetry Pouch

Single Patient Use Only • Disposable  
Rx Only

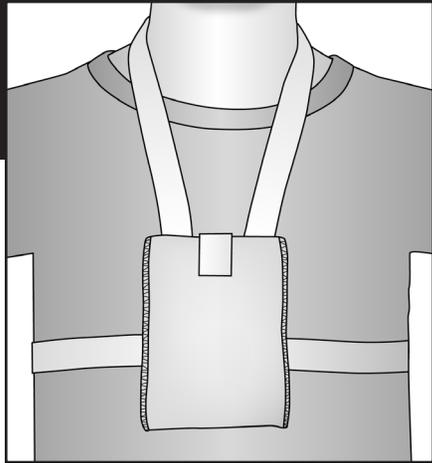

### IMPORTANT INFORMATION

Please read all instructions before use. Correct application is essential to proper functioning of products.

### INTENDED USE

Pouch for telemetry unit.

### PRECAUTIONS

- The physician or practitioner is responsible for providing use instructions and precautions to other healthcare practitioners, care providers involved in the patients' care and the patient.
- Follow good nursing and basic patient care practices and other recommended practices of your health care facility.
- Follow the instructions of physician (or properly licensed practitioner) for length and duration of use.

### SIZING


Universal

### CAUTIONS

- Do not expose this product to burning cigarettes or any source of flame
- Secure straps properly without over tightening.

### INSTRUCTIONS FOR USE

1. Position telemetry unit securely inside the pouch so that the lead wires extend from the top.
2. Place pouch in proper position on patient (as directed for the use of the telemetry monitoring device), bring upper strap around patient's neck (as pictured), and close with hook and loop.
3. If using a telemetry pouch equipped with a waist strap, bring side strap around patient's trunk and attach to the lower hook and loop closure. **NOTE:** Excess material from strap may be cut if necessary.

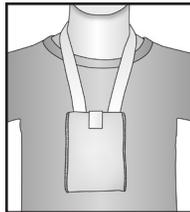


### CARE INSTRUCTIONS

This product is disposable and should not be laundered or reused. DISCARD AFTER USE.




Manufactured by DeRoyal  
200 DeBusk Lane  
Powell, TN 37849 U.S.A.  
888.938.7828  
865.938.7828

Part #0-610  
©2008 DeRoyal. All Rights Reserved.  
Revised 2/08

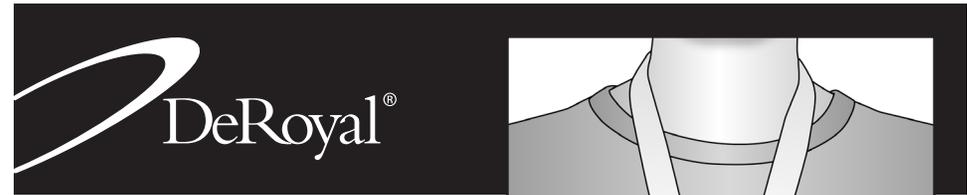

## Telemetry Pouch

Single Patient Use Only • Disposable  
Rx Only

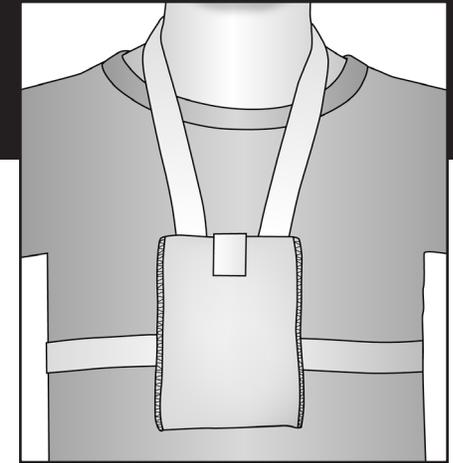

### IMPORTANT INFORMATION

Please read all instructions before use. Correct application is essential to proper functioning of products.

### INTENDED USE

Pouch for telemetry unit.

### PRECAUTIONS

- The physician or practitioner is responsible for providing use instructions and precautions to other healthcare practitioners, care providers involved in the patients' care and the patient.
- Follow good nursing and basic patient care practices and other recommended practices of your health care facility.
- Follow the instructions of physician (or properly licensed practitioner) for length and duration of use.

### SIZING


Universal

### CAUTIONS

- Do not expose this product to burning cigarettes or any source of flame
- Secure straps properly without over tightening.

### INSTRUCTIONS FOR USE

1. Position telemetry unit securely inside the pouch so that the lead wires extend from the top.
2. Place pouch in proper position on patient (as directed for the use of the telemetry monitoring device), bring upper strap around patient's neck (as pictured), and close with hook and loop.
3. If using a telemetry pouch equipped with a waist strap, bring side strap around patient's trunk and attach to the lower hook and loop closure. **NOTE:** Excess material from strap may be cut if necessary.

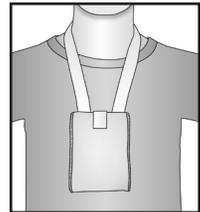

### CARE INSTRUCTIONS

This product is disposable and should not be laundered or reused. DISCARD AFTER USE.



Manufactured by DeRoyal  
200 DeBusk Lane  
Powell, TN 37849 U.S.A.  
888.938.7828  
865.938.7828

Part #0-610  
©2008 DeRoyal. All Rights Reserved.  
Revised 2/08