## Explanation of Symbols

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## Explication des symboles

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## Verklaring van de symbolen

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Contains or presence of natural rubber latex

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Application videos are available on [www.deroyal.com](http://www.deroyal.com).

### SIZING

Universal

### CLEANING AND/OR MAINTENANCE

Prior to washing, remove any detachable rigid stays, fasten any buckles and tie any laces to minimize damage during wash and dry cycles. Close any hook and loop fasteners before washing to help prevent lint build up and pilling. For maximum product life and to help prevent any straps from tangling, wash in a mesh laundry bag. Hand or machine-wash with a mild fabric detergent in warm water up to 60°C / 140°F. Bleach as directed on container. Do not iron. Lay flat and open and allow to air dry, or tumble dry on low heat for up to 40 minutes. Inspect the product after washing to ensure it is intact and replace any previously removed stays. Hook and loop fasteners may collect lint after repeated use or laundering, reducing grip strength. As needed, use a stiff bristle brush to remove lint from the "hook" side. Test hook and loop adhesion before use and discard product if hook and loop does not hold securely. During the washing, metal buckles could bend and plastic quick-release buckles could crack or break. Always ensure that all buckles are in good working order and are not bent, cracked or broken before each use. If the product is damaged, do not use it and replace it with a new one. Customers could expect up to 5% shrinkage in cotton materials over the life of the product.

### WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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Improving Care. Improving Business.™

Manufacturer:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or 865.938.7828  
[www.deroyal.com](http://www.deroyal.com)

QNET BV  
Hommerterweg 286  
6436 AM Amsternrade  
The Netherlands

### Part #0-701G

Revised 2-2015

©2015 DeRoyal Industries, Inc.  
All Rights Reserved.

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## FOAM WHEELCHAIR BELT

**RX ONLY  
SINGLE PATIENT USE  
NON-STERILE  
NOT MADE WITH NATURAL RUBBER LATEX**

### IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

### INTENDED USE

The DeRoyal Foam Wheelchair Belt is intended to be prescribed by a properly licensed practitioner to be used to help keep the patient positioned in the wheelchair and prevent forward sliding and getting up unassisted.

### CONTRAINDICATIONS

- Do not use on a patient who is or may become highly aggressive or agitated.
- Never use on a patient if an I.V. or wound site could be compromised by the device.

### WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Always secure straps to a portion of the bed or wheelchair that moves with the patient.
- Always keep straps out of patient's reach.
- Belts should be worn over patient's clothing.
- Check the patient regularly to ensure circulation is not compromised.

### PRECAUTIONS

This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use. Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

### INSTRUCTIONS FOR USE

- Wrap the belt around the wheelchair back and the patient and secure hook and loop closure. For the self-releasing feature, make sure the hook and loop closure is in front of the patient. For a more restrictive device, make sure the hook and loop closure is behind the wheelchair back.
- Do not over tighten. Two fingers should fit between the device and the patient to ensure circulation is not compromised.

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