



Improving Care. Improving Business.®

VESTS

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

DeRoyal® Vests are intended to be used as a physical reminder for the patient to remain in the bed or wheelchair and for postural support.

CONTRAINDICATIONS


- Do not use on a patient who is or may become highly aggressive or agitated.
- Never use on a patient if an I.V. or wound site could be compromised by the device.

WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Always secure straps to a portion of the bed that moves with the patient. Do not secure straps to bed or gurney side rails.
- Always keep straps out of patient's reach.
- Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure breathing and circulation are not compromised.

CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this product must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Inspect product for damaged or missing components before use.
- Discontinue use and consult your prescriber if the product or its components break or become damaged, or cleaning is no longer sufficient to sanitize product.
- Vest should be worn over patient's clothing. Consult prescriber immediately if patient experiences sensation changes, unusual

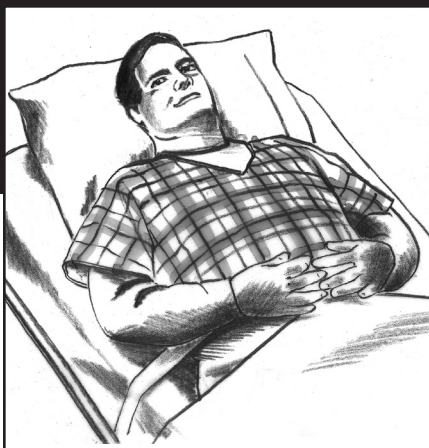

reactions, swelling or increased pain while using this product. Discontinue use if pressure injuries develop.

- Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

INSTRUCTIONS FOR USE

1. Place the patient's arms through the sleeves or sleeve openings. **NOTE:** Patient should be sitting up while vest is being applied.
2. Secure zipper or hook and loop closure in back. **NOTE:** Do not overtighten. Two fingers should fit between the device and the patient to ensure circulation is not compromised.
3. Secure the straps to the portion of the bed frame or the wheelchair that moves with the patient using quick-release ties or quick-release buckles. Please refer to the instructions below.

SHOULDER STRAP APPLICATION (NOT AVAILABLE ON ALL STYLES)

If applying shoulder straps, take the end of the strap and feed it through the loop located on the shoulder portion of the vest. Feed the end of the strap up and over through the teeth of the buckle. Pull the strap completely through until it engages the buckle. Apply the other shoulder strap following the same instructions. Apply the straps to a portion of the head of the bed that moves with the patient using quick-release ties. Follow the quick-release ties application instructions below.

QUICK-RELEASE TIES

Wrap the strap(s) around the anchor once. Take the loose end of the strap(s) and create a loop. Cross the loop over the other strap(s) that is wrapped around the anchor and feed it through opening as if tying a shoelace. Hold that loop and create another loop with the loose strap(s) and feed it through the first loop. Pull the final loop and the strap(s) attached to the cuff in opposite directions to form a quick-release tie.

QUICK-RELEASE BUCKLES

Release the female end of the buckle and wrap its strap around the anchor. Pull the female portion through the loop and connect the male and female ends of the buckle. Pull the excess strap to tighten.



Improving Care. Improving Business.®

-sizing

SIZE*	COLOR	BODY WEIGHT	CHEST CIRCUMFERENCE
x=XS X-Small	Pink	up to 110 lbs	up to 32"
x=S Small	Red	100 - 130 lbs	30" - 34"
x=M Medium	Green	120 - 160 lbs	33" - 38"
x=L Large	Yellow	150 - 185 lbs	36" - 40"
x=XL X-Large	Navy	175 - 220 lbs	39" - 43"
x=XXL XX-Large	Black	200 - 250 lbs	41" - 45"
x=XXXL XXX-Large	Brown	230 lbs and up	45" and up

*Use patient's body weight and chest circumference to determine proper vest size

PEDIATRIC VEST SIZING

Small 2-4 years

Medium 4-6 years

Large 6-8 years

CLEANING AND/OR MAINTENANCE (IF APPLICABLE)

Please refer to hospital approved guidelines for cleaning and disinfecting of the device.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufacturer:

DeRoyal Industries, Inc.

200 DeBusk Lane

Powell, TN 37849 USA

888.938.7828 or 865.938.7828

www.deroyal.com

Part#0-711 Revised 2/19

©2019 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.