REGGI-ARTO

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LIMB HOLDERS

RX ONLY
SINGLE PATIENT USE
NON-STERILE
NOT MADE WITH NATURAL RUBBER LATEX

IMPORTANT INFORMATION
Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE
The DeRoyal Limb Holders are intended to be prescribed by a properly licensed practitioner to be used for patients at risk of unplanned extubations and patients who disrupt medical treatment.

CONTRAINDICATIONS

- Do not use on a patient who is or may become highly aggressive or agitated.
- Never use on a patient if an I.V. or wound site could be compromised by the device.

WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Secure straps to a portion of the bed or gurney that moves with the patient. Do not secure straps to bed or gurney side rails.
- Keep straps out of patient’s reach. Patient should not be able to remove device.
- Make sure bed or gurney side rails are up while product is in use.
- Check the patient regularly to ensure circulation is not compromised.

PRECAUTIONS
This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient’s care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

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Be sure to follow your facility’s policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

- Wrap the limb holder around the patient’s wrist or ankle, ensuring that the cuff closure is on the outside of the device.
- Secure the hook and loop closure and if applicable, the buckle closure.
- Two fingers should fit between the device and patient to ensure circulation is not compromised.
- Secure the strap(s) using quick-release ties or quick release buckles depending on the style of the limb holder. Please refer to the step-by-step instructions for Quick-Release Ties and Quick-Release Buckles.
- For limb holders with two straps, the two-point attachment option is available. One strap can be attached near the head of the bed and the other strap near the foot of the bed using quick release ties or buckles.
- Secure excess strap away from patient’s reach.

Application videos are available on www.deroyal.com.

QUICK RELEASE TIES

Wrap the strap(s) around the anchor once. Take the loose end of the strap(s) and create a loop. Cross the loop over the other strap(s) that is wrapped around the anchor and feed it through opening as if tying a shoelace. Hold that loop and create another loop with the loose strap(s) and feed it through the first loop. Pull the final loop and the strap(s) attached to the cuff in opposite directions to form a quick release tie.

QUICK RELEASE BUCKLES

Release the female end of the buckle and wrap its strap around the anchor. Pull the female portion through the loop and connect the male and female ends of the buckle. Pull the excess strap to tighten.

CLEANING AND/OR MAINTENANCE

To clean, hand wash with a mild fabric detergent in warm water. Do not bleach. Do not iron. Lay flat and open and allow to air dry. Inspect the product after washing to ensure it is intact. If the product is damaged, do not use it and replace it with a new one.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL’S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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	
Improving Care. Improving Business.™	
	
Manufacturer: DeRoyal Industries, Inc. 200 DeBusk Lane Powell, TN 37849 USA 888.938.7828 or 865.938.7828 www.deroyal.com	Hommerterweg 286 6436 AM Arnhemrade The Netherlands
Part #0-717G	
Revised 12/14	
©2014 DeRoyal Industries, Inc. All Rights Reserved.	
	9/2014

	
Manufacturer: DeRoyal Industries, Inc. 200 DeBusk Lane Powell, TN 37849 USA 888.938.7828 or 865.938.7828 www.deroyal.com	Hommerterweg 286 6436 AM Arnhemrade The Netherlands
Part #0-717G	
Revised 12/14	
©2014 DeRoyal Industries, Inc. All Rights Reserved.	
	9/2014

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