



Improving Care. Improving Business.®

## SECURITY CUFFS

	<b>SINGLE PATIENT USE</b>
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>



### IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

### INTENDED USE

The DeRoyal® Security Cuffs are intended to be prescribed by a properly licensed practitioner to protect the patient from harming themselves, other patients, family members and/or staff.

### CONTRAINDICATIONS

Never use on a patient if an I.V. or wound site could be compromised by the device.

### WARNINGS

- Keep straps and buckles out of patient's reach. Patient should not be able to remove device. Improper application or use of any restraint may result in serious injury or death.
- Inspect product for damage, missing components, or contamination before use. Ensure that all hook and loop and buckle closures function properly.
- Use hospital approved bed safety guidelines. If securing the device to a bed always secure straps to the portion of the bed that moves with the patient. Do not secure straps to bed side rails.
- Check the patient regularly per hospital's policies and procedures to ensure circulation is not compromised.

### CAUTIONS

- This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and cautions to other healthcare practitioners or healthcare providers involved in the patient's care.
- Consult physician (or properly licensed practitioner) immediately if patient experiences sensation changes, unusual reactions, swelling or increased pain while using this device. Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

### INSTRUCTION FOR USE

#### SECURING THE CUFFS TO THE BED:

1. Find a location on the bed frame that moves with the patient as specified by hospital bed safety guidelines.
2. Unfasten the hook and loop and/or quick release buckles from the bed attachment portion of the strap.
3. Secure the bed attachment portion with hook and loop and/or quick release buckles to the bed frame.

#### SECURING THE CUFFS ON PATIENT

1. Unfasten all hook and loop and/or quick release buckles from the cuff closure.
2. Place the cuff under the patient's limb.
3. Wrap the cuff around the patient's limb and secure all hook and loop and/or quick release buckles.
4. Insert the hook side into the anchor or slot and bring it around to secure it to the loop.
5. Once all security cuffs are applied to the bed or gurney, secure the cuffs on the patient.
6. Wrap the neoprene cuff around the patient's wrist or ankle with the loose neoprene side first and then follow with the neoprene side with hook. Secure the loop side of the cuff and follow with the other hook side of the cuff.

#### CLEANING AND/OR MAINTENANCE (IF APPLICABLE)

Please refer to hospital approved guidelines for cleaning and disinfecting of the device.

#### STORAGE AND TRANSPORT CONDITIONS

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

#### WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Improving Care. Improving Business.®



Manufactured by:  
DeRoyal Industries, Inc.

200 DeBusk Lane

Powell, TN 37849 USA

888.938.7828 or (001) 865.938.7828

[www.deroyal.com](http://www.deroyal.com)

Part#0-869 Revised 6/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.