

NON-ADHERENT WOUND DRESSING

# MULTIPAD™

MULTIPAD™



- Highly absorbent
- Non-adhering
- Non-linting
- Multi-layered



## NON-ADHERENT WOUND DRESSING

# MULTIPAD™

### FEATURES & BENEFITS

- A multi-layered, highly absorbent wound dressing
- Gentle, non-woven rayon pad between two non-adhering polypropylene contact layers
- Will not stick to wound site, damage fragile granulation tissue or traumatize wound upon removal
- Non-linting
- Absorbs significantly more drainage than similar dressings resulting in fewer dressing changes
- Use in place of gauze and tape when more absorbency is needed

### INDICATIONS

- Dermal ulcers
- Abdominal wounds
- Infected wounds
- Superficial wounds
- Lacerations, cuts and abrasions
- Donor sites
- 2nd degree burns
- Diabetic ulcers



Clinical Photography by Etris Associates, Philadelphia, PA

### APPLICATION & REMOVAL

1. Peel open package.
2. Position Multipad over wound.
3. Secure with tape, APEX™, or Stretch Net™ Tubular Elastic Bandage.
4. To remove, gently lift dressing from wound. Clean and redress wound if necessary.
5. Dispose of soiled dressing per institution's policy.

Product #	Size	Qty/Bx	Qty/Cs
46-011	2" x 2"	25	200
46-012	4" x 4"	25	200
46-013	4" x 8"	25	100
46-014	7½" x 7½"	10	50

TO PLACE AN ORDER OR FOR MORE INFORMATION

PLEASE CALL: 800251-9864

888.938.7828 (WITHIN USA)

865.938.7828 (OUTSIDE USA)

800.543.2182 (FAX)



200 DeBusk Lane  
Powell, Tennessee 37849 U.S.A.  
[www.deroyal.com](http://www.deroyal.com)

Reprint #0-883 • ©2005 DeRoyal.  
All Rights Reserved.  
Rev. 5/2005