


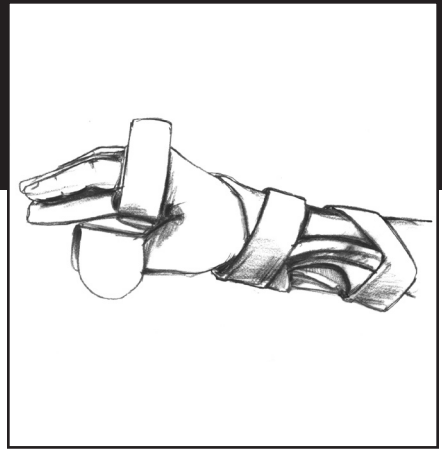





RESTING HAND SPLINT

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.



IMPORTANT INFORMATION

 Please read all instructions, warnings and cautions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® Resting Hand Splint is intended for post surgery/injury positioning, wrist drop or finger flexor tightness.

CONTRAINDICATIONS




Not recommended when moderate to severe spasticity is present.

CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the wrist (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber

if the brace or its components break or become damaged.

- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

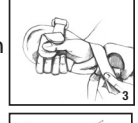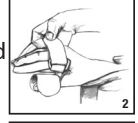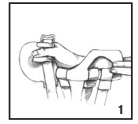

SIZING

Splint size is determined by the width of the hand at the metacarpophalangeal joints.

SIZE	WIDTH
B	2 ¼" - 2 ¾" (5.71 - 6.98 cm)
C	2 ⅞" - 3 ¼" (7.28 - 8.26 cm)
D	3 ⅜" - 3 ¾" (8.57 - 9.53 cm)

INSTRUCTIONS FOR USE

1. To assess fit, position splint on the palm of the hand. Determine the proper thumb and wrist position. Remove splint and bend the splint to obtain the desired position.
2. To secure the finger strap, reposition the adjusted splint on the hand. Hold the folded edge of the strap and gently pull it through the D-ring and fold to contact the hook material.
3. For secure wrist support, this splint is



designed with a 2-piece strap system. Strap #1 starts distal to the head of the ulna, crosses the wrist diagonally and adheres to the proximal portion of the hook and loop closure. Strap #2 crosses strap #1 and adheres to the distal portion of the hook and loop closure.

4. To secure the forearm strap, wrap strap around the forearm and attach to the hook material on the splint. Trim excess strap.

CARE INSTRUCTIONS

Straps may be removed and laundered separately. Hand wash with a mild soap and let air dry completely before reapplication.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for ninety (90) days from the date of shipment from deroyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

PART#1-125 | REVISED 6/2023

©2023 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal and the DeRoyal logo are trademarks or registered trademarks of DeRoyal Industries, Inc.