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Manufactured by:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
www.derooyal.com

EMERGÓ EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands





Part#1-503G  
Revised 2/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.

**SIZING**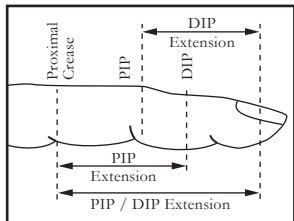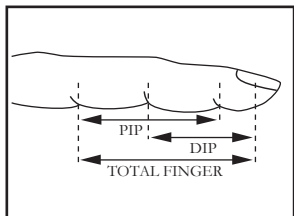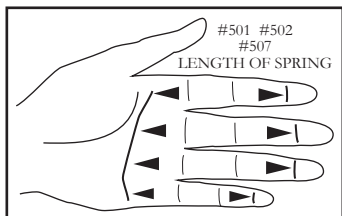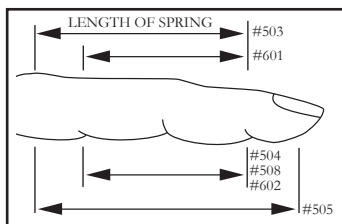

**ENGLISH**

**FINGER SPRING**  
**SPRING COIL FINGER FLEXION ASSIST**

SINGLE PATIENT USE	
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

**IMPORTANT INFORMATION**  
Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any product.

**INTENDED USE**  
The DeRoyal® Finger Spring fits from the base of the proximal phalanx to the distal interphalangeal joint. The coil springs, at the lateral aspect of the proximal interphalangeal joint (PIP), act to increase PIP flexion while allowing active extension. The distal phalanx is free to allow functional use of the finger pad.

- CAUTIONS**
- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this product must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
  - Use only as directed by medical professional.
  - Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the product to build muscle strength (2) regular physical therapy exercises to maintain the strength of the finger (3) a limitation on duration of use or (4) some combination of these.
  - Inspect product for damaged or missing components before use.
  - Discontinue use and consult your prescriber if the product or its components break or become damaged.
  - Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
  - Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Discontinue use if pressure injuries develop.
  - Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

**SIZING**  
To determine splint length, measure distance on dorsal surface as indicated in the diagram.

SIZE	LENGTH
A	2½” 6.3 cm
B	2¾” 6.9 cm
C	3” 7.6 cm
D	3¼” 8.2 cm

- INSTRUCTIONS FOR USE**
- To fit the splint, slide splint onto finger so the proximal pad rests over the top of the proximal phalanx and the coils lie lateral to the proximal interphalangeal joint.
  - The distal dorsal pad should lie proximal to the distal interphalangeal joint.
  - To secure the splint, pull the strap through the D-ring to hold the splint on the proximal phalanx.
  - Place the tab of hook material close to the D-ring and fold the strapping down onto loop material.
  - Cut excess strapping as needed.

**CARE INSTRUCTIONS**  
Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

**WARRANTY**  
DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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