

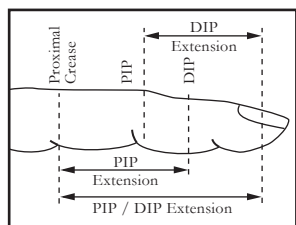
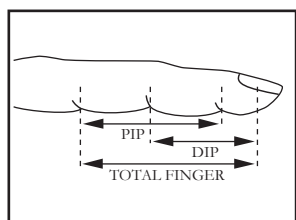
SIZING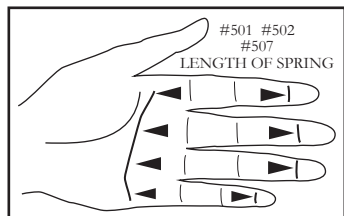

Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (601) 865.938.7828
www.derooyal.com

EMERGEO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands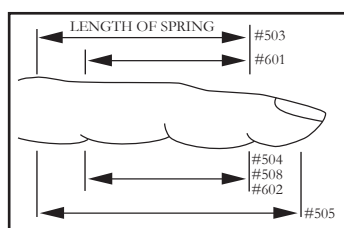

Part#1-507G
Revised 2/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.

**FINGER SPRING
PROXIMAL INTERPHALANGIAL EXTENSION ASSIST**

SINGLE PATIENT USE	
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any product.

INTENDED USE

The DeRoyal® Finger Spring fits from the head of the metacarpal to the distal interphalangeal joint. It is used to assist the proximal interphalangeal joint (PIP) into increased extension. It has a minimal effect on metacarpalphalangeal (MP) extension.

CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this product must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the product to build muscle strength (2) regular physical therapy exercises to maintain the strength of the finger (3) a limitation on duration of use or (4) some combination of these.
- Inspect product for damaged or missing components before use.
- Discontinue use and consult your prescriber if the product or its components break or become damaged.
- Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

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SIZING




To determine splint length, measure involved finger from palmar crease to distal finger crease when finger is extended.

SIZE	LENGTH	
AA	2¼"	5.7 cm
A	2⅝"	6.6 cm
B	3"	7.6 cm
C	3⅞"	8.5 cm
D	3¾"	9.5 cm

INSTRUCTIONS FOR USE

FITTING THE SPRING (A)

- Hold the spring at the proximal and distal pads. **NOTE:** The proximal pad has the white plastic covered wire attached to it.
- Slide splint onto finger so the proximal pad rests over the volar aspect of the metacarpal head and the distal pad rests at the distal interphalangeal joint (DIP).

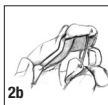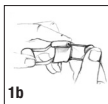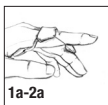ADJUST SPRING TENSION (B)

- To increase the force of extension, spread the volar pads apart before placing on the finger.
- To decrease the force of extension, pinch the volar pads together before placing on the finger.

CARE INSTRUCTIONS

Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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