## HAND SPLINT, METACARPAL PHALANGEAL, FLEXION

## FÉRULA DE MANO, METACARPOFALÁNGICA, FLEXIÓN

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## HAND SPLINT, METACARPAL PHALANGEAL, FLEXIÓN

	SINGLE PATIENT USE
	<b>NON-STERILE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>
<b>RX ONLY</b>	<b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b>

### IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

### INTENDED USE

The DeRoyal® Hand Splint dynamically flexes metacarpalphalangeal (MP) joints and is intended to address tightness or tendency to extend MP joints following wrist fracture.

### CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the hand (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

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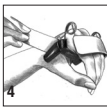
### SIZING

To determine the appropriate size, measure the width of the hand at MP joint in inches or centimeters.

SIZE	LENGTH	
B	2¼ - 2¾"	5,7 - 7 cm
C	2⅞ - 3¼"	7 - 8,25 cm
D	3⅜ - 3⅞"	8,25 - 9,5 cm

### INSTRUCTIONS FOR USE

- Slip hand between palmar bar and MP flexion bar so the palmar bar fits into the palmar crease.
- The dorsal MP flexion bar should rest over the proximal phalanges.
- Pull strap up and over the back of the hand to obtain the desired tightness. Secure the orthosis with the hook and loop closure material.

### CARE INSTRUCTIONS

Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

### STORAGE AND TRANSPORT CONDITIONS

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

### WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>

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