**MANUFACTURED BY:**  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
www.deroyal.com

**MedEnvoy Global B.V.**  
Prinses Margrietplantsoen 33 – Suite 123  
2595 AM the Hague  
The Netherlands

**UK Responsible Person**  
MedEnvoy UK Limited  
85, Great Portland Street, First Floor  
London, W1W 7LT  
United Kingdom

**PART#363178G I REVISED 10/23**

©2023 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, DeRoyal logo, Puddle Guppy are trademarks or registered trademarks of DeRoyal Industries, Inc.

## EN: PUDDLE GUPPY™ FLOOR SUCTION DEVICE

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## EN: ENGLISH

### PUDDLE GUPPY™ FLOOR SUCTION DEVICE

|                |   |
|----------------|---|
|                | <b>SINGLE PATIENT USE</b>   |
|                | <b>NON-STERILE</b>  |
|                | <b>MEDICAL DEVICE</b>   |
|                | <b>NOT MADE WITH NATURAL RUBBER LATEX</b>   |
| <b>RX ONLY</b> | <b>FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.</b> |

#### IMPORTANT INFORMATION

Please read carefully all instructions, warnings, and precautions before use. Correct setup is essential for proper functioning of product.

#### INTENDED USE

The DeRoyal® Puddle Guppy™ is intended to aid in the management and disposal of waste fluid during a variety of medical procedures.

#### WARNINGS

Fluids that enter the system should not be re-introduced to the body. This system is not recommended for pleural drainage.

#### PRECAUTIONS

- Ensure all connections are secure. Improper set up may result in product malfunction or loss of suction.
- DO NOT** reuse the Puddle Guppy for multiple medical/ surgical procedures. The Fish Stick may be reused pursuant to the cleaning guidelines outlined below.
- When handling the Puddle Guppy, utilize universal personal protection equipment following local, state, regional, or national/federal guidelines to ensure protection from hazardous fluids.

#### DIRECTIONS FOR USE

- Attach standard suction tubing to the suction port on any suction canister.
- Attach the other end of the suction tubing to the port on the top of the Puddle Guppy.
- Place the Puddle Guppy on the floor, flat side down. Place it within a puddle of fluid or directly under the stream of fluid.
- If it is necessary to move the Puddle Guppy, insert the notched end of the included "Fish Stick" into the keyhole on the top of the Puddle Guppy.
- Dispose of Puddle Guppy and suction tubing in compliance with facility's guidelines.

#### CLEANING

The Fish Stick is reusable in patient care settings. If the Fish Stick becomes dirty, cleanse with appropriate cleaning agents:

- Any household cleaner is appropriate. A 10% bleach solution or a 10% isopropyl alcohol solution will clean the Fish Stick without damaging the plastic.
- DO NOT** use acetone-based cleaners as they may damage the Fish Stick.

#### STERILIZATION INFORMATION

**NON-STERILE PRODUCT:** DeRoyal intends that non-sterile products for use in sterile environments be further processed by our customers, including further packaging or sterilization according to their own validated processes. These products are to be sterilized by ethylene oxide only, as other sterilization methods have not been validated by DeRoyal and may damage the product, which could cause a device malfunction or injury to the patient

#### STORAGE AND TRANSPORT CONDITIONS

Do not store product at extreme temperatures or in a moist/damp environment.

|  |                                |
|--|--------------------------------|
|  | <b>KEEP DRY</b>                |
|  | <b>KEEP AWAY FROM SUNLIGHT</b> |

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

#### WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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