



## Solid Side ShieldZ™

Disposable • Do Not Reuse  
Non-Sterile • Rx Only



### Important

Read carefully and completely before using or applying.

### Intended Use

The Solid Side ShieldZ are plastic solid add-on fluid protectors for use with prescription and other glasses. Either clip-on or slip-on, they are designed to reduce potential exposure against splashes, spatters and droplets of blood or other bodily fluids.

### Indications


The DeRoyal Solid Side ShieldZ is intended for use by clinicians whenever splashes, spray, spatters or droplets of blood or other bodily fluids may be generated and contamination may be reasonably anticipated.

### Precautions

- Destroy after a single use.
- This device is NOT designed, sold or intended for use except as indicated.
- This product is NOT impact resistant and should not be worn as protection from solid objects.
- Do NOT use as a shield against acids, alkalines and other hazardous chemicals known to attack polyester and other films.
- Do NOT use as protection against a strong light source, gases or vapors of any kind, sparks, radiation, or ultraviolet rays.
- Do NOT use while smoking or performing any activities involving the use of or proximity to flame or heat-producing equipment.

### Directions For Use

1. After removal from package, orient Side ShieldZ such that larger space from hole to edge of ShieldZ is toward lens of glasses and solid piece of plastic is on outside of glasses.

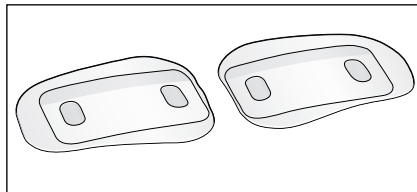

2. Weave arm of glasses through both holes of ShieldZ.
3. Alternatively, clip the arms of glasses between two pieces of plastic of the ShieldZ.

### Warnings

- Wearing this product will NOT eliminate exposure or risk of contracting any disease or infection.
- Change this product immediately and dispose of properly if damaged or contaminated with blood or bodily fluids.
- This product does NOT meet ANSI Standard Z87.1-1989.



### Manufacturer

DeRoyal  
200 DeBusk Lane, Powell, TN 37849 U.S.A.  
888.938.7828

### Authorized Representative

DeRoyal Europe, Ltd.  
Virginia Road, Kells, County Meath Ireland  
353.46.80100

©2003 DeRoyal. All Rights Reserved.  
Part #4-10608 Revised 1/03