



Improving Care. Improving Business.®

DRAIN-JUG™ CONTAINER

	DO NOT REUSE
	STERILIZED WITH ETHYLENE OXIDE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read carefully all instructions, warnings, and cautions before use. Correct setup is essential for proper functioning of product.

INTENDED USE

The Drain-Jug™ container is a 15-Liter gravity fed container that allows quick and easy collection of fluids for disposal when a floor drain is not present.

CONTRAINDICATIONS

- Not intended for long-term storage of bodily fluids.
- Fluids that may enter the system should not be reintroduced into the body.

WARNINGS

- Inspect the container for visible cracks or broken components. The rim should be smooth and free of any burrs to ensure proper function. Replace at healthcare professional's discretion.
- **DO NOT** reuse or attempt to sterilize.

CAUTIONS

- Ensure all connections are secure. Improper setup may result in product malfunction.
- Avoid tilting. The product should be maintained on a level surface.
- All measurements are approximate.
- When handling the Drain-Jug™ container, utilize universal personal protective equipment including gloves, face mask, and proper clothing to ensure protection from splashes during transport and disposal of aspirated fluid in the container.

INDICATIONS

Use of this device is indicated when large volumes of fluid are being collected from the following: cystoscopic pouches, arthroscopic pump drains, and gravity drains.

DIRECTIONS FOR USE

1. Snap on Drain-Jug™ container lid at the “6:00” position. Seal container along opposite sides simultaneously until both hands meet at “12:00” position for final seal.
2. Place Drain-Jug™ container beneath fluid source.
3. Insert tubing or attach hose to Drain-Jug™ container port.



DIRECTIONS FOR DISPOSAL

1. When full, disconnect tubing and cap off container.
2. Transport full Drain-Jug™ container to disposal area with standard ring stand or dolly.
3. To empty contents quickly and safely, utilize DeRoyal® Siphon Pump or DeRoyal® Aqua Box fluid waste disposal unit to move waste fluid into hopper.
4. If utilizing solidifier, tubing adapter can be removed for easy application of solidifier powder through Drain-Jug™ container portal.
5. Dispose of drainage and disposable container in compliance with local, state, and federal requirements.

STORAGE AND TRANSPORT CONDITIONS

DO NOT store product at extreme temperatures or in a moist damp environment.

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.



Improving Care. Improving Business.®



Improving Care. Improving Business.®



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#401701 Revised 7/2020

©2020 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care. Improving Business and Drain-Jug are trademarks or registered trademarks of DeRoyal Industries, Inc.