



Infant Transport Mattress

HNICU-200

Non-sterile

IMPORTANT




The following information should be read prior to use of this device.

INTENDED USE

The Infant Transport Mattress helps in maintaining an infant's core body temperature during transport. The non-toxic gel in the mattress absorbs unwanted vibration.

DIRECTIONS FOR USE

1. Squeeze Infant Transport Mattress until inner bag bursts and chemical reaction begins.
2. Knead mattress to evenly disperse contents. For warmer nurseries (75°F/24°C & up) or premature baby, knead pack after activation for 30 seconds.
3. Place infant on mattress during transport.

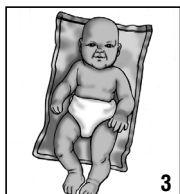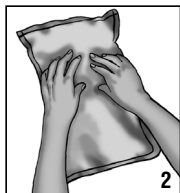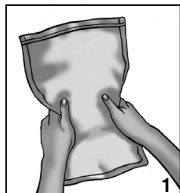

NOTE: Temperature 105°F/41°C (from 75°F/24°C). Contents are non-toxic.

PRECAUTIONS

Activate away from face and infant. Per hospital procedure, closely monitor infant's skin for redness or irritation. Monitor neonate's core body temperature.

WARNINGS

Use carefully. Check infant regularly, particularly those who are premature. If irritation develops, discontinue use. Do not puncture bag. If contents are swallowed, consult physician immediately.

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).



Manufacturer

DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828

Authorized Representative

DeRoyal Europe, Ltd.
Virginia Road, Kells, County Meath Ireland
353.46.80100

Part #45-00084

Revised 3/00

©2000 DeRoyal. All Rights Reserved.

Single Patient Use