
Improving Care. Improving Business.<sup>®</sup>

Improving Care. Improving Business.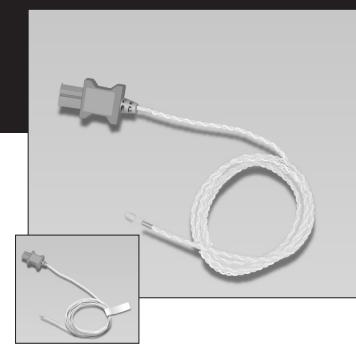

Manufactured by:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
[www.deroyal.com](http://www.deroyal.com)

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#### Part#74-14908

Revised 8/2022

©2022 DeRoyal Industries, Inc.  
All Rights Reserved. DeRoyal, the DeRoyal logo and  
Improving Care. Improving Business. is a registered  
trademark of DeRoyal Industries, Inc.

## ENGLISH

### NASOPHARYNGEAL TEMPERATURE PROBE

	<b>DO NOT RESTERILIZE</b>
	<b>DO NOT REUSE</b>
	<b>STERILIZED WITH ETHYLENE OXIDE</b>
	<b>NOT MADE WITH NATURAL RUBBER LATEX</b>

**IMPORTANT**  
 Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury.

**INTENDED USE**  
The DeRoyal<sup>®</sup> Nasopharyngeal Temperature Probe is to be used for routine monitoring of core body temperature.

#### PRODUCT DESCRIPTION

Probe is inserted nasally. It is sterile, individually packaged, and comes in different sensor types. It has a low-friction surface finish to improve ease of insertion. The temperature sensor, beneath the distal tip, is highly accurate and designed to be compatible with the temperature monitor using the appropriate DeRoyal<sup>®</sup> reusable interface cable.

#### CONTRAINdications

- The use of this device in the nasopharynx cavity may be contraindicated in patients with esophageal diverticulum or stenosis, in neonates or small infants undergoing neck surgery and in patients undergoing a tracheotomy or insertion of an internal jugular catheter.
- Nasopharyngeal insertion is not recommended for patients who are taking anticoagulants or when manipulation of mucosa is undesirable.
- DO NOT** use rectally.

#### WARNINGS

- Not tested for use in MR environments. Use in an MR environment may cause burn to patient or other serious injury.

- Inspect packaging for any damage and product for any damage, contaminate or missing parts prior to use.
- While adverse reactions are rare, the following have been reported to be associated with the use of temperature probes during insertion or while the device was in use:
  - airway obstruction
  - aspiration pneumonitis
  - bronchial insertion
  - electrical burns
  - epistaxis
  - esophageal abrasion
  - esophageal perforation
  - tracheal insertion
  - trauma to pharynx
- May cause epistaxis (nosebleed). In the pregnant patient, children with large adenoids, and patients with clotting disorders, the incidence of epistaxis is high.
- The mucosal temperature can vary based on exact location (i.e., anterior versus posterior mucosa) and respiratory cycle (i.e., inspiration versus expiration). The exact location and timing of the respiratory cycle should be noted and taken into consideration when using a nasopharyngeal temperature probe.

#### CAUTIONS

- Lubricate the probe with a medical grade water-soluble lubricant before insertion and use accepted medical techniques during insertion and removal of the probe.
- Take care to ensure that the cable and connector do not get wet and that proper techniques are used during electrosurgical procedures to reduce radio frequency interference current and potential burns to the patient.
- An adequate electrosurgical dispersive ground electrode close to the active surgical site should be properly connected.
- DO NOT** intertwine the cables, especially the monitor cables, with the electrosurgical unit's cables.
- The operation of the patient temperature monitor may be temporarily affected during electrosurgical activations. Unusual temperature readings should be checked.
- If probe has inaccurate, unstable, or no temperature readings, discard and replace.
- The nasopharyngeal probe is intended to be used with DeRoyal-approved interface cables. Use with

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an incompatible cable may affect performance.

#### MR COMPATIBILITY

• The DeRoyal<sup>®</sup> Temperature Probe has **NOT** been evaluated for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. The safety of the Probe in the MR environment is unknown. Scanning a patient who has this medical device may result in patient injury.

#### DIRECTIONS FOR USE

- Remove the Nasopharyngeal Temperature Probe from its sterile package.
- Verify the compatibility of the probe, interface cable, and patient monitor.
- Lubricate the probe tip with a suitable water soluble lubricant.
- Insert the probe and guide it through the nasal cavity into the nasopharynx cavity.
- Connect the temperature probe to the appropriate reusable temperature cable from DeRoyal. Connect the cable to the patient monitor. Secure the cable using drape clip.
- Follow the directions for use of the patient temperature monitor instrument.
- Disconnect the probe (at the connector) and discard the probe. To disconnect, grasp both connectors firmly and pull. **DO NOT** pull on cable or wire.
- This probe is **NOT** reusable.
- Discard according to facility protocol.

#### OPERATING SPECIFICATIONS

- Rated output range: 25° to 45° Celsius
- The reference body site is the core body temperature.
- The measuring site is the nasopharynx.
- Accurate to +/- 0.2° Celsius
- Time response:
  - The temperature probe has a direct mode of operation.
  - The maximum heating transient time is 40 seconds.
  - The maximum cooling transient time is 30 seconds.

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#### STORAGE AND TRANSPORT CONDITIONS

	<b>KEEP DRY</b>
	<b>KEEP AWAY FROM SUNLIGHT</b>
	<b>DO NOT USE IF PACKAGE IS DAMAGED</b>
	<b>RELATIVE HUMIDITY: 85% NON-CONDENSING</b>
	<b>STORAGE TEMPERATURE: -25°C TO +55°C</b>
	<b>OPERATING TEMPERATURE: 25°C TO 45°C</b>

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

#### WARRANTY

DeRoyal<sup>®</sup> products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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