DEROYAL® MANIFOLDS AND STOPCOCKS

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Manufacturer: DeRoyal Industries, Inc. 200 DeBusk Lane Powell, TN 37849 USA 888.938.7828 or 865.938.7828 www.deroyal.com

QNET BV Hommerterweg 286 6436 AM Amstenrade The Netherlands

Part #884213G

Revised 9/14

©2013 DeRoyal Industries, Inc. All Rights Reserved.

- WARNINGS
All connections should be hand tightened, without over-tightening, to ensure their security and prevent damage to the device.

RELEASE NOTE: All pressure ratings are based on DeRoyal's sterilization cycle. Other sterilization may affect product performance.

Table with 3 columns: Product, Maximum Pressure (PSI), Maximum Pressure Rating (KPa). Rows include Low Pressure, Medium Pressure, High Pressure, and Small catheters.

INTENDED USE
DeRoyal® manifolds and stopcocks are intended to be used during cardiac catheterization procedures for intra-arterial and intravenous administration of contrast, saline or radiographic contrast media.

DO NOT REUSE
R1 ONLY
DO NOT REUSE
DEROYAL® MANIFOLDS AND STOPCOCKS

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STERILIZATION INFORMATION
Non-sterile product: DeRoyal intends that non-sterile products for use in sterile environments be further processed by our customers, including further packaging or sterilization according to their own validated processes.

STORAGE CONDITIONS
Do not store product at extreme temperatures or in a moist/damp environment.

PRECAUTIONS
Product should not be used if sterile packaging is damaged or opened.

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