ONE-PIECE EXTRICATION COLLAR Single Patient Use Only • RX Only

IMPORTANT INFORMATION: Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of product.

INTENDED USE: The DeRoyal® One-Piece Extrication Collar is intended to be used for short term cervical immobilization for extrication.

PRECAUTIONS: This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to the patient as well as other healthcare providers involved in the patient's care. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

CARE INSTRUCTIONS: Wipe down with damp cloth. Air dry.

WARRANTY: DEROYAL PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

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
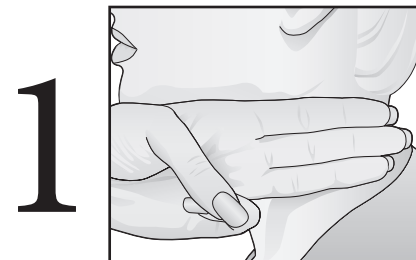
1 With patient in neutral alignment, use fingers to measure distance from the top of the shoulder to the bottom of the chin.

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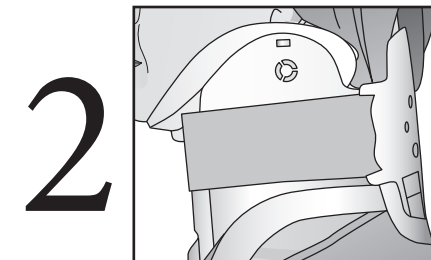2 Place fingers on the lowest size line and select size closest to the height of your fingers.

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APPLICATION • APLICACIÓN • MISE EN PLACE • APLICAÇÃO

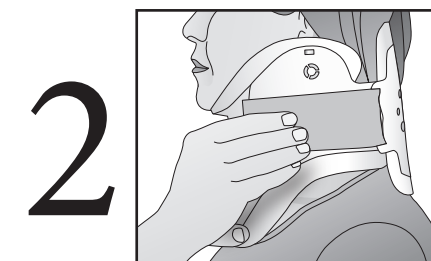
1 With patient in neutral alignment, slide back portion of collar behind patient's neck until loop strap can be seen.

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
2 Wrap to the front of the neck and position the chin support.

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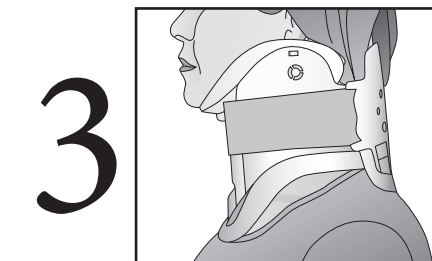3 Secure with hook and loop closure.

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Fixe com o fecho com velcro.

Scan for further information and instructional videos. Please visit our website at www.deroyal.com.

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 **DeRoyal** Improving Care. Improving Business.

Manufactured by DeRoyal Industries, Inc. • 200 DeBusk Lane • Powell, TN 37849 U.S.A. • 888.938.7828 • 865.938.7828 ©2012 DeRoyal Industries, Inc. All Rights Reserved. Revised 9/12 • www.deroyal.com